

# AREBOS

## Shower Chair

AR-HE-BH60004



Follow all safety precautions in this user manual to ensure safe use.



Thank you for your trust in AREBOS.



Please read and save these instructions. Read through this user's manual carefully before using this product. Protect yourself and others by observing all safety information, warnings and cautions. Failure to comply with instructions could result in personal injury and/or damage to product or property. Please retain instructions for future reference.

## Safety instructions

### Symbol explanation



This product must not be disposed of with household waste

Symbol	Introductions	Symbol	Introductions
	Batch Code		Catalogue number
	Warnings and Precautions		non-sterile
	medical device		Manufacture Date
	Manufacturer Name Address		Unique Device Identification
	CE Symbol		Model Number
	Country of Manufacture		Importer
	European Authorized Representative		

### Safety Instructions

- Read the instructions completely before first use. Unpack the product and check that all components are included in the package and that the product may be damaged in transit. If this is the case, please contact the service address below.
- There is a special risk of slipping when sitting down and getting up in damp bathtubs or shower rooms, especially if bath products or detergents are used.
- After each assembly, make sure that the stability of your bath stool is given.
- The risk of tipping can be increased by balancing body movement.
- If you cannot keep yourself sufficient, ask an assistant for help.
- Do not use high pressure cleaners or harsh, corrosive chemicals for cleaning.

**Warning! Risk of injury!**

- Incorrect use of the bath stool can cause injury.
- Never stand on the shower stool!

**Warning! Risk of damage!**

- Incorrect use of the bath stool can cause damage to the product.
- Do not use the shower stool if the product is damaged or if the screw connection is loose.

**The shower stool should not be used by persons with**

- cognitive disorders
- Strong balance disorders
- Seat inability

## Scope of delivery

- Seating surface
- 4 chair legs (telescopic)
- 4 rubber feet
- Screw

## Assembly



### Height adjustment of the chair legs

- By adjusting the height of the chair legs, the seat height of the person can be individually adjusted. To do this, you have to press the spring button on the leg of the chair against the telescopic tube. Then slide the tube into the desired position.



- **Warning!** After adjustment , make sure the spring button clicks into one of the existing holes. **Risk of falling!**
- Check that all chair legs are set at the same height.

## Technical Specifications

Material	Aluminum/HDPE
Color	Silver/White
Seat width	approx. 50 cm
Seat depth	approx. 30 cm
Seat	Adjustable up to approx. 52 cm
Weight	2 kg
Max. load capacity	100 kg

## Cleaning and care

- We recommend cleaning the shower stool after each use. You can wipe the bath stool with a damp cloth, in case of heavier soiling, you can use mild detergents. To disinfect the product, you can use a standard surface cleaner.

### Maintenance

- The shower stool is designed to be maintenance-free. If the shower stool is defective, contact the specialist dealer to replace the product.

## Disposal Instruction

### Disposal of the packaging

- Please make reference to the guidelines and standards for appropriate disposal of the packaging valid in your region. In part, the package may consist of plastic bags - watch this respect, with special care to ensure that this is not out of the reach of children. There is a risk of suffocation!

### Disposal of waste equipment

- Equipment must be disposed of in accordance with the rules and regulations of the local waste disposal.

Meaning of the “dustbin”



Protect our environment; electrical appliances do not belong in household waste. Use the provided for the disposal of electrical equipment collection points and enter your electrical and electronic equipment that you no longer use. They help ensure that the potential effects of incorrect disposal on the environment and human health to be avoided. So, do your part to recycle, recycling and other forms of recovery of waste electrical and electronic equipment . Information on where the devices are disposed of, please contact your local authorities or local Governments .from your municipalities or municipal administrations.

### [Importer]



Company: Canbolat Vertriebs GmbH  
Address: Gneisenaustraße 10-11, 97074 Würzburg, Germany

**Our customer service number: +49 (0) 931-45232700**

Canbolat Vertriebs GmbH • Gneisenaustraße 10-11 • 97074 Würzburg

## EU Declaration of Conformity

We, the

Canbolat Vertriebs GmbH, Gneisenaustraße 10-11, 97074 Würzburg, Germany

SRN: DE-IM-000044811

Hereby declare that the product named below, seen its design and construction as well as according to our sales, has been complied with the relevant and basic health.

Product Name: AREBOS Shower Chair

Item No.: 4260199756560

Model Number: AR-HE-BH60004

Basic UDI-DI:697353654showerchairD7

Classification: Class I, According to Rule 1,

Annex VIII, Regulation (EU) 2017/745

If the device is modified without our consent, this declaration of conformity loses its validity.

Tested acc. to:

Annex II+III of Regulation (EU) 2017/745

EN ISO 14971:2019

EN ISO 15223-1:2021

EN ISO 20417: 2021

EN ISO 10993-1:2020

EN ISO 10993-5:2009

EN ISO 10993-10: 2023

EN ISO 10993-23: 2021

Date/Signature Manufacturer/Location:

Würzburg, 07.11.2024



Signature:

Dipl.-Inform. (Univ.) Korhan Canbolat, Managing Director

Representative of this Instruction Manual/Technical Data:

Dipl.-Inform. (Univ.) Korhan Canbolat, Managing Director

Office Address:

Canbolat Vertriebs GmbH

Gneisenaustraße 10-11

97074 Würzburg

**Our customer service number: +49 (0) 931-45232700**

Canbolat Vertriebs GmbH • Gneisenaustraße 10-11 • 97074 Würzburg

Return address can be found in the imprint: <https://www.arebos.de/impressum/>

Sales tax identification number: DE 263752326

The court of the commercial register entry is Würzburg, HRB 10082 WEEE-Reg.-Nr. DE 61617071